

Special Feature	Hypoallergenic
Colour	Teal blue
Size	45 x 38 x 12
Fill Material	Cotton
Shape	U shaped semi-circle
Material	Faux Leather, Cotton
Item Firmness Description	Semi Hard
Product Care Instructions	Hand Wash Only

- **Efficient Ergonomic Design** - This positioner is shaped like a U letter with two covers for patients' legs. They are ergonomically designed and user-friendly because the U shape is high-efficient according to dynamics.
- **Protect Patients' Health** - As we know, bedridden patients possibly get worse if they don't receive proper nursing.
- **Durable & Soft Materials** - The positioner adopts PU material and EPE cotton. The inner thickened material is EPE cotton, which is soft and comfortable for the patients and not easy to deform.
- **Simple Use Steps** - Because their usage is convenient, even the aged can easily use this product with the following four steps. Firstly, you should put it under the patient's legs. Secondly, stand it up between the legs.
- **Extensive Application** - The products are widely used in many common scenarios for easy operation. The hospital can adopt the devices to nurse the patients after surgery.
- **Patient Turning Device Aid - Equipment to Help Patients Turn Over**
Turning patients on a regular basis is essential to prevent the formation of new wounds and allow existing wounds to heal. Our turning device makes this easy, and the turning pillow can be used to change the patient's changing pads, mattresses, and help defecation.
- **IDEAL SOLUTIONS for ASSISTED TURNING CARE BEDRIDDEN**
Easily change postures, Relieve the suffering of patients.
Swings left and right, Reduce the pressure on your hips and back.

Correct the bedridden person's position, Avoid muscle atrophy caused by long-term bed rest.

Helps relieve in the back, Hips, Legs or knees.

More effective leg support, Improve blood circulation to the legs.

Lying on your side should align your spine.

Professionals suggest that people who are bedridden for a long time turn over every 2 hours

- **PATIENT TURNING DEVICE**

EASY TO OPERATE: Put the upper part of the knee into the two grooves of the turn over pillow, One hand holding the handle, And the other hand supporting the patient's waist or back, Easy pulled.

- **MULTIFUNCTIONAL TURNING DEVICE**

The turning device is specially designed for transferring patients and the elderly. It can assist the patients and the elderly to perform simple bed-resting exercises and correct the spine, and can avoid the aggravation of muscle atrophy caused by long-term bed-rest. This turning pillow is a good helper for nursing people.

- **Easy to fix the position**

Fix one end of the fixing strap on the bed handle to facilitate staff to help patients roll over, change clothes, or change sheets, etc

- **Help the Patient Turn Over**

The caregiver only needs to push the patient's shoulder, and pulling the side straps on the turning pillow to easily help the patient to turn over.

- **Comfortable Material**

Invisible zipper ;

Removable pillowcase ;

soft PU leather - waterproof, hypoallergenic, and prevents the skin of the elderly from being worn.

- **High-Density Remember Sponge**

The built-in high-density foam plastic has good cushioning performance, no deformation, no compression, strong support.

- **Patient Turning Device Bed Rest**

The Upgraded Turning Pillow features U-shaped grooves to ensure the patient's legs remain comfortable. The thicker sponge forms the largest possible contact force area with the body, so as to avoid the patient's feeling of pressure and discomfort.

The Elderly Turning Pillow is for the patient to turn over to meet their needs (eg diaper changing, massage, body cleaning, exercise).

The turning pillow can also be placed upside down between the legs, allowing people in wheelchairs or those who need a walker to keep their knees apart and prevent joint deformation or hardening.

- Patient Turning Device Multifunctional Turnover Device U-Shaped Bed Rest Turning Pillow for Patients or Bedridden Elderly People Lying in Bed
- The surface is soft Faux leather environmentally friendly and hypoallergic prevents abrasion of the skin of the elderly
- More line with ergonomic curved surface design The curved surface design can be easily turned over, reducing the difficulty of single person nursing and the burden on patients
- Change the posture in bed to avoid bedsores Auxiliary exercise, promote blood circulation Reduce excretion and nursing difficulties caused by physical discomfort, and reduce nursing burden
- Bedridden Turning Device, Anti-Decubitus Care Pad Bed Wedges, Nursing Tool, Bed Rest Products

The surface layer is made of soft PU leather and anti-allergic, and prevents the elderly from being scratched. Some parts of the knees are placed in the two grooves of the product and turned over. The back is a high-quality injection sponge. It will not deform after a long time. A certain degree of flexibility can prevent elderly people from being injured or uncomfortable. It is easy to pull the two handles at both ends to help the elderly turn over. Bed exercise helps prevent muscle atrophy. Reduce the difficulty of faecal care and reduce the burden on the family. Material: High-elastic injection sponge. The multifunctional is suitable for elderly patients who are and bedridden for a long time. The multifunctional turning pillow nursing pad is suitable for elderly patients who are and stay in bed for a long time.